

SETPOINT SYSTEM USE & CARE



CHARGING CHARGER

- Always place your Charger on the Docking Station when you're not wearing it to keep it fully charged.
- The Charger is fully charged when its LED turns solid green, or when the Docking Station's LED is solid blue.
- Be sure your Charger is fully charged before bringing it to the clinic.



CLEANING CHARGER

- Clean your Charger with a dry, lint-free cloth, avoiding the magnetic latch.
- To remove lotion or oils, you can use isopropyl alcohol (IPA) wipes—these are also safe for cleaning the magnetic latch area.
- Never spray liquids directly on the Charger or submerge it in any liquid.
- Keep the magnetic latch area free of lint and dirt.



CHARGING IMPLANT

- Wear your Charger around your neck and ensure the magnetic latch is securely closed.
- Charging starts when the LED blinks green or orange, and you hear three beeps that rise in tone.
- Charging is complete when the LED turns solid green and you hear four beeps—three rising tones followed by a repeat of the last tone.
- To stay on track, it's recommended to charge your Implant at the same day and time each week.



TRAVELING


- Always use the Carrying Case to transport your Charger.
- Do not use your Charger while traveling in any vehicle, including cars, trains, boats, or planes.
- If airport security has questions about your SetPoint System, show them your Patient ID Card.



Avoid areas marked with radio-frequency (RF) safety warning signs.



Contact your clinic immediately if you experience any pain or discomfort.

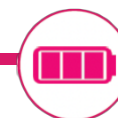

Remember to charge your Implant every week



This information is not a substitute for fully reading and understanding the Patient IFU (Instructions for Use). Please refer to the Patient IFU for complete details on how to use the SetPoint System safely and effectively. In addition to reading this guide, please watch the training video [here](#) or scan the QR Code.

